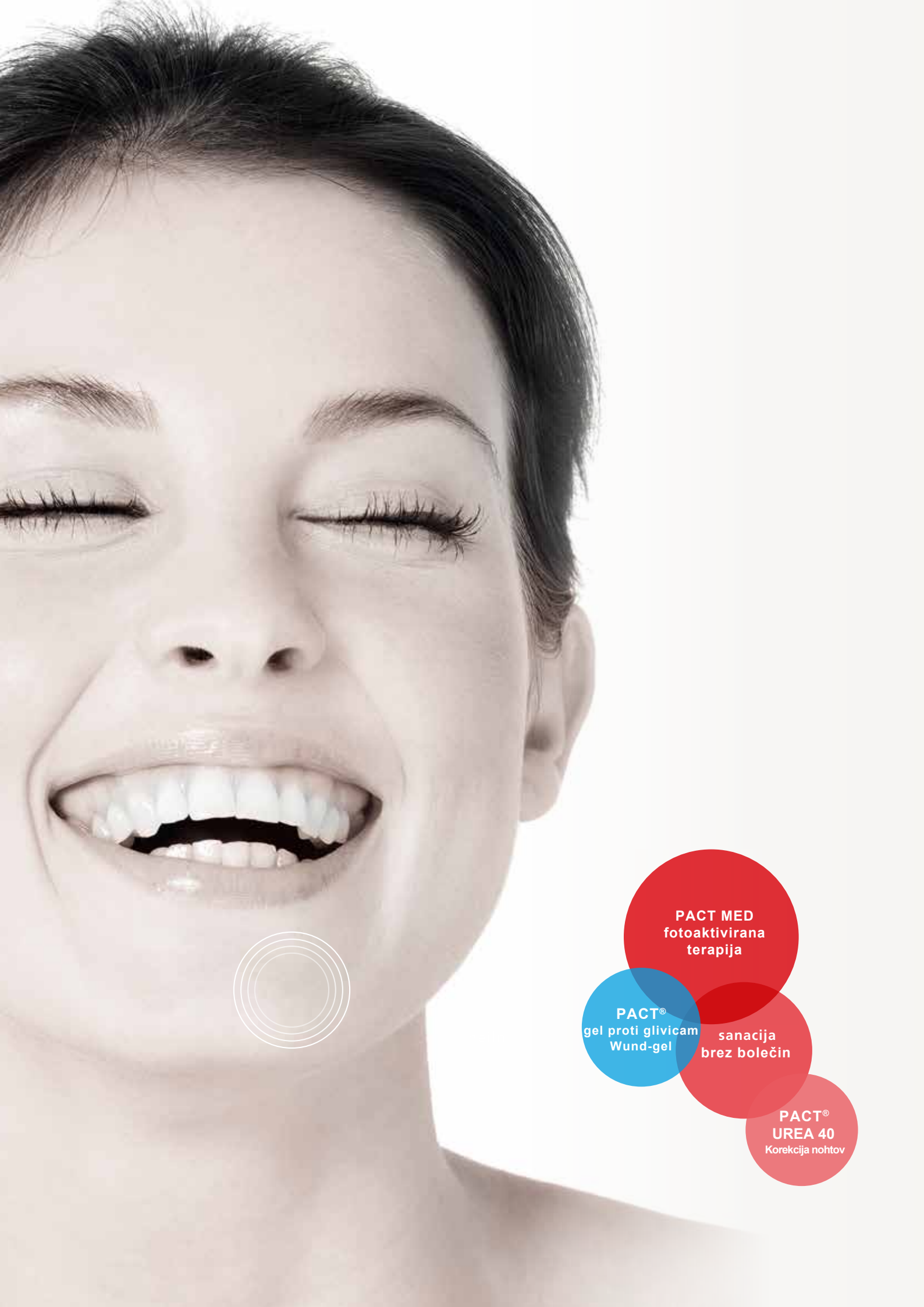
FOTOAKTIVIRANA TERAPIJA PRI INFEKCIJAH

For infections

**Photo-activated
Therapy**

- PACT MED**
fotoaktivirana
terapija
- PACT®**
gel proti glivicam
Wund-gel
- sanacija
brez bolečin
- PACT®**
UREA 40
Korekcija nohtov

Photoaktivirana terapija

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Vaš Hahn Medical Systems Team

Photo-activated therapy is indispensable in modern medicine since the 1990s. For the treatment of tumors, skin diseases and wound infections.

Since the middle of the 90s Univ.-Prof. Dr. Rainer Hahn has been working on photosensitizers and compatible light sources. Dr. Frank Schynowski is one of the most experienced chemist in the field of PACT®. Hahn Medical Systems has developed to a leading company in this field of medicine.

PACT®: made in Tübingen, Germany.

You can rely on our know-how and the efficacy of PACT®.

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PACT® Antifungal NailGel

The PACT® Antifungal NailGel was specially developed for application to toenails and fingernails and it only works in combination with exposure to light from a PACT® MED light device. Other light sources or red light must not be used with PACT® Antifungal NailGel.

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**99,99 % of all
bacteria are killed.
Without any side
effects. No
resistances.**

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PACT[®] WoundGel

With PACT[®] WoundGel you can treat superficial infections and burns, in combination with the PACT[®] MED phototherapy device.

A rapid healing is supported by a moist environment. PACT[®] WoundGel in combination with PACT[®] MED can also be used for the treatment of infected wounds under supervision of a medical expert.

- Photosensitizer Gel for PACT[®] Wound Healing Therapy (30 ml)
Art. No. 1001766

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Art.-Nr. 1070200

PACT® UREA 40

To reduce nail discolourations and to dissolve brittle nail material, for example due to nail fungus infections. PACT® UREA 40 (urea approx. 40%) can dissolve the brittle (or fungal infected) part of the nail. The damaged part of the nail can be detached and cautiously cut off or removed with a nail file.

- PACT® UREA 40 ointment, incl. package information leaflet, 5 m plaster, 1 x nail file

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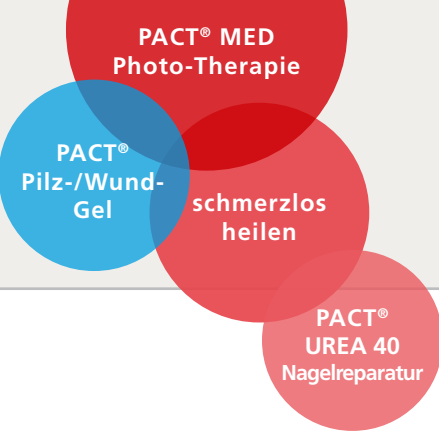
The used photosensitizer tolonium chloride has been proven safe for the therapy of poisonings for a long time. Primarily such patients attracted attention by getting sensitised by light. From this the photo-activated therapy with tolonium chloride was developed.

Under irradiation with the PACT[®] photo-therapy devices PACT[®] MED, the applied PACT[®]-sensitizer attacks, after short residence time, the cell walls of bacteria and fungi. Thereby, PACT[®] is also effective against antibiotic-resistant bacteria as well as deep under the surface.

After therapy the dye is rinsed off without residue. Nothing hinders wound healing. Regeneration is promoted.

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Broad spectrum bactericide

PACT® often replaces local antibiotic therapy, also in case of antibiotic-resistant germs.

PACT® can sustainably inactivate and dissolve biofilms, reduces viruses and fungi without discoloration, without any side effects.

PACT® improves the treatment of bone infections and ulcers.

PACT® improves the treatment of bone infections and ulcers.

PACT® can be used easily and economically.

PACT® can be delegated to the assistant.

PACT® has over 10 years of experience, countless scientific studies.

PACT® is recommended by German and international experts.

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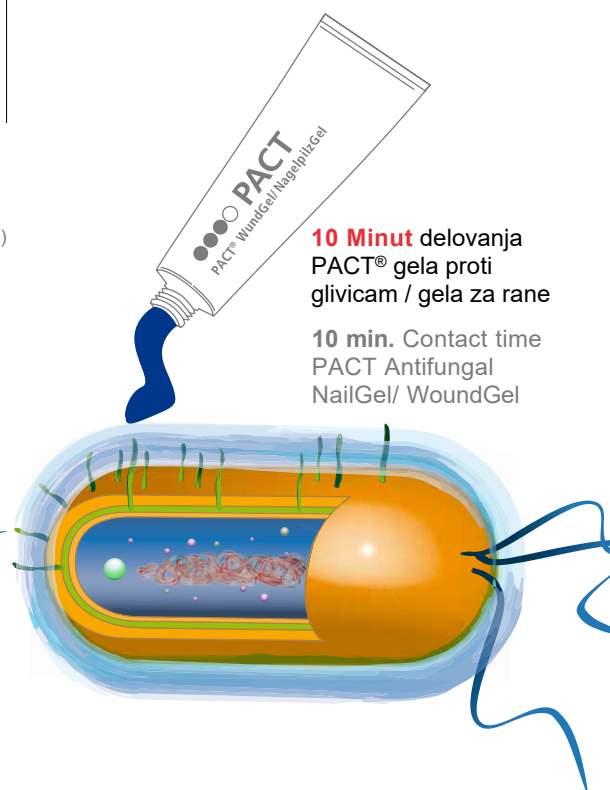
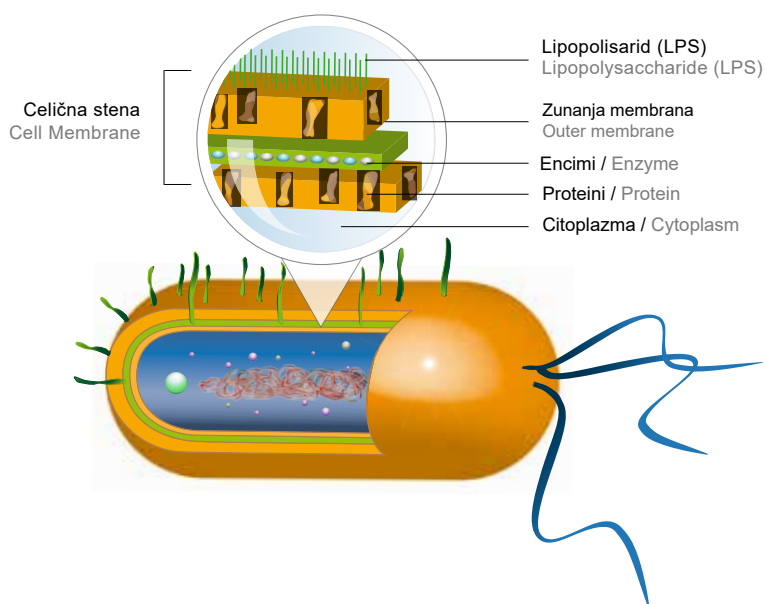
Ground State PACT[®]

Photosensitizer diffuses into biofilms and pockets.
Interacts with cellwalls.

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
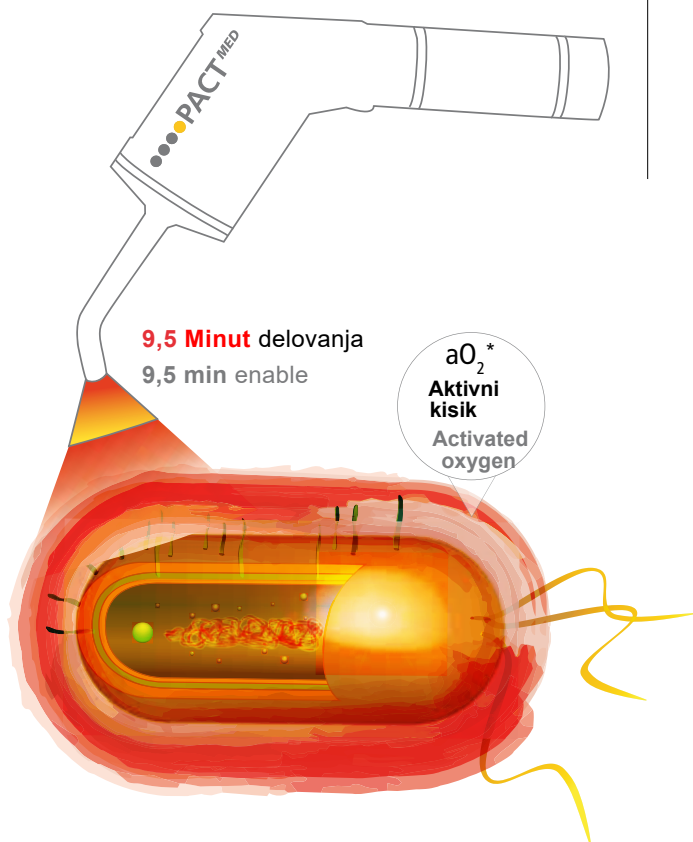
99,99% killing rate

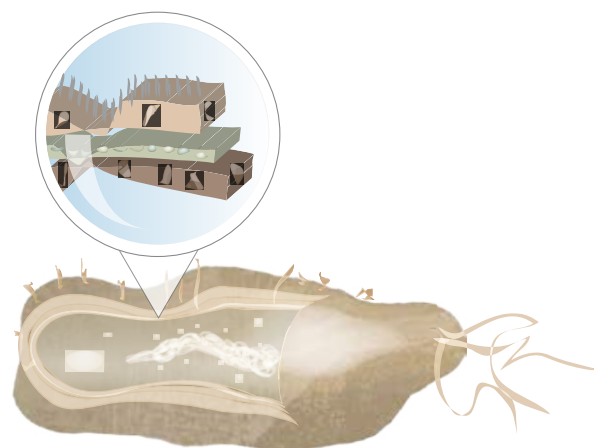of all bacteria, fungi and viruses by destroying their cell walls with aO_2^* . Even multi-resistant germs. Spares somatic cells (cell membrane).

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During PACT® treatment 99.99% of all bacteria and fungi are killed. Without side effects. No resistances.

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Pridržujemo si pravico do tiskarskih napak.

© Copyright Hahn Medical Systems GmbH

Informacije veljavne od januarja 2016

Note:

The information concerning the Cumdente products is based on extensive research and application-specific experience. We reproduce these results to the best of our knowledge. Technical modifications in the product development and errors are, however, reserved. When using the products, the user should observe the recommendations and information.

The price-list is a component part of this catalogue. Reproduction of this catalogue, even in extract form, is prohibited without our express consent. Errors and omissions excepted.

© Copyright Hahn Medical Systems GmbH

Information as of Januar 2016